

	<b>Diplomat Dental s.r.o., Vrbovská cesta 17, 921 01 Piešťany, Slovak Republic</b>		
	<b>Change description:</b> New FW 1.1.13 Model Pro	<b>Change is valid for:</b> Model Pro 500, 600, 700, 800	<b>Technical information</b> <b>No: 006</b>
			<b>Change is valid from:</b> <b>24.06.2023</b>

## New FW 1.1.13 – Model Pro

### Changes:

- Added control of micro motor LED light power via the service menu of an instrument
- Improved communication of wireless foot controller
- Added signalization of foot controller pairing on the RF receiver PCB
- Improved stand by mode of wireless foot controller
- FW preparation for adding recipro mode for micro motors
- FW preparation for new SWING foot controller
- Improved water bottle calibration system
- (bugfix) Unit configuration sometimes changed after FW update
- (bugfix) Some setting were moved between instruments while double-work activated

### Breakdown of separate FW packages for individual PCBs in FW 1.1.13

TB+CB PREM_R	CB - 1.0.0.24, TB - 1.0.0.1C
SB_PREM1_R	SB - 1.0.0.18
FC_PREM_TKC2	FC Cable - 1.0.0.D
FC_PREM_TUC2	FC Cable - 1.0.0.D
FC_PREM_TKW1	FC_RF - 1.0.0.D
FC_PREM_TUW1	FC_RF - 1.0.0.D
FC_PREM_R	FC_RX - 1.0.0.8
AB_PREM2	AB - 1.0.0.7
CHES_GREAT	CHES - 1.0.1.A
CS_PREM	CSB - 1.0.0.3

It is recommended to update all Model Pro dental units to the newest FW.