

	<b>Diplomat Dental s.r.o., Vrbovská cesta 17, 921 01 Piešťany, Slovak Republic</b>		
	<b>Change description:</b> Bottle filter	<b>Change is valid for:</b> All units with hygiene programs	<b>Technical information</b> <b>No: 007</b>
			<b>Change is valid from:</b> <b>15.07.2023</b>

## Bottle filter

Distilled water bottle filter (pictured below) is to be replaced for a new one every 6 months as a disinfectant solution can gradually block the filter.

If there is a visible drop of pressure when using distilled water before this interval, its recommended to replace the filter as soon as possible.

Illustrative picture:

